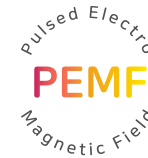
ikeun™

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Growth Plate Stimulator

Thank you for purchasing ikeun™

- For proper and safe use, please read user manual carefully.
- After reading the manual, please keep it in a location where you can check it at any time.

• Product Features	14
• Mode Settings	15
• How to Use	16
• After Use	17
• Charging Methods	18
• Precaution for Use	19
• FAQ	20
• Product Specification	21

Product Features

iKeun™ is a growth plate stimulator integrated with MN's patented technology.

Patent Name

Key growth auxiliary device by pulsed micro-electro magnetic field and driving method thereof.

[Patent No. : 10-2104871]

[PCT No. : KR2020/005565]

Patent Name

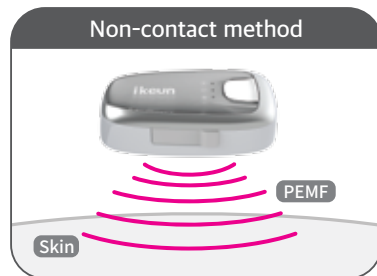
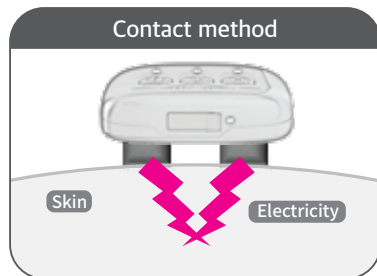
Growth plate stimulation device for chair mounting & driving method thereof.

[Patent No. : 10-2240304]

[PCT No. : KR2021/005426]

※ iKeun™ is not classified into medical device category in Korea.

iKeun™, containing pulsed electromagnetic field (PEMF) and quad coils, stimulates growth plate of knee area of growing children. iKeun™ provides comfortable fit since it can be worn directly on clothes.

Mode Settings

Mode	Sleep Mode (Single Action)	Intensive Mode (Dual Action)
Explanation	PEMF massage <ul style="list-style-type: none"> · PEMF only · Night time (10 PM to 2 AM) · Does not interfere with good sleep. 	PEMF + Vibration massage <ul style="list-style-type: none"> · PEMF + Vibration · Day time · Can be used when studying or taking a break.
Mode Indicator Lamp		
Function	PEMF 9Hz	PEMF 28Hz vibration
Operation Time	4 hours	1 hours
Recommended Frequency of use	Once a day	Once a day

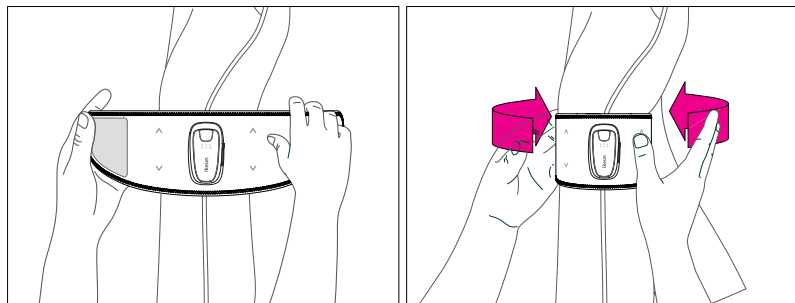
※ Operation time may vary depending on the environment.

※ Do not use continuously as it may cause malfunction.

※ In sleep mode, it doesn't vibrate and you may hear a fine pulse sound.


1 How to wear

- ① Wear Ikeun™ so that the device fits tightly around the knee.
 - ② After wearing the device around the knee gently fix it around using the velcro band.
- ※ Can be worn over clothing.

2 How it works

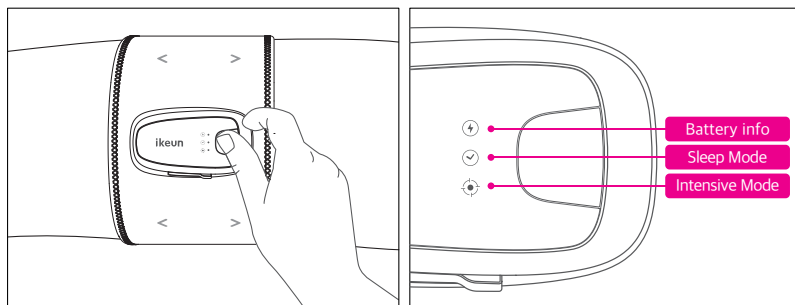- ① If you press the power button, the orange LED will light and the intense mood will be activated.
- ② You can select sleep or intense mode each time you press the button.

- ※ When each mode is used, the power is automatically turned off.
- ※ If you want to stop the operation during use, press the button for more than 3 seconds.

After use

- ① Turn off the power.
 - ② Wipe off any dirt on the Ikeun™ with a lightly wet cloth, then wipe off the moisture with a dry cloth.
 - ③ Please keep it in Ikeun™ pouch.
- ※ Do not use alkaline detergents, alcohol, thinner, benzene, or acetone.
- ※ This product is not waterproof. Do not immerse in water or wash with running water.

Storage

- When you are not using the device for a long period of time, clean and store the device. Avoid storing in the following locations.
 - Places with temperature of 35°C or higher or temperature of 10°C or lower.
 - Places exposed to direct sunlight.
 - Places within reach of infants or pets.
 - Places exposed to flammable substances such as chemicals or gases.
 - Places with a risk of adverse effects such as air pressure, humidity, ventilation, salt, etc.

Battery info

- This product has a built-in lithium polymer battery.
- Batteries are consumables products. As a result, the usable time of the product will gradually decrease if the battery is charged for a certain period.
- Replacement service will not be charged during the warrant period.

Charging Methods


Lithium polymer batteries are installed in ikeun™.
When using the product for the first time, fully charge the battery before use.

Charging time	3 hours
Usage time	About 4 hours for intensive mode, about 16 hours for sleep mode

- ※ The operating time may vary depending on the operating mode, operating level, and environment.
- ※ Charging time may vary depending on the remaining battery capacity and environment.

Charging method

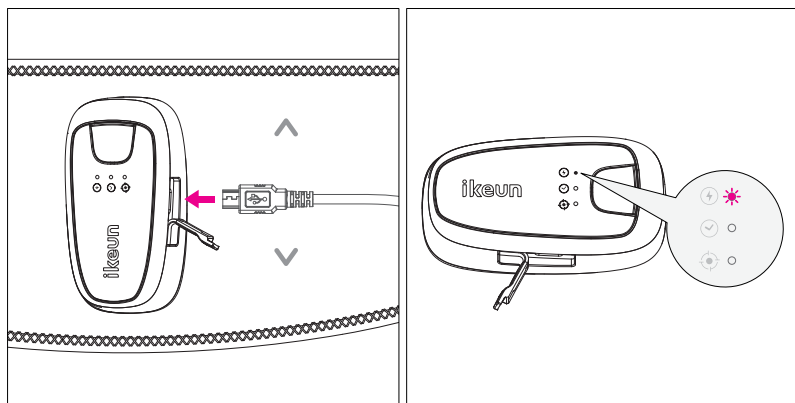- ① Open the cover of the charging port of the product and connect the charger on a Y-type gender (connection).
- ② Connect the Y-type gender (connection) to the USB charging port.
- ③ When charging is completed, separate the gender (connection) from the USB port and the device. Then use or store it.

- ※ The device does not operate while battery charging, so be sure to use it after charging.
- ※ The device displays green when charging is completed.

Precaution for Use


Before use, read the 'Precautions for use' thoroughly before use.

- Since this device is not a medical device, it is not recommended for treatment purposes.
- When using the product for the first time, fully charge it before use.
- Avoid using it for purposes other than those described in the manual.
- Do not disassemble, repair, or modify the device.
- Do not heat the product or apply severe pressures.
- Do not damage the Velcro band, process it, bend, pull, twist, or tie the band with force.
- Dusts or substances on the Velcro band may cause it not to adhere well, so keep it clean.
- Contact with a conductive substance (metal, liquid, etc.) on the connecting part of the charger may cause a malfunction, so be careful not to get dust, etc. into the charging terminals.
- This product is not waterproof. Be careful not to get water inside.
- Be sure to turn off the power when not in use or when you store it.
- When you store the device, avoid places with high humidity, temperatures of 10°C or lower or with temperatures of 35°C or higher.
- Do not charge the product during use.
- If you have any of the following problems, stop using the product and ask a specialist.
 - Those who experience health issues such as red spots, swelling, itchiness, or marks during use.
 - Those with sensitive skin due to skin diseases or allergies.
 - Those who have undergone surgery due to a knee injury.
 - Those with skin issues such as wounds, eczema, or boils.
 - Patients with malignant tumors (including sarcoma).
- If you notice any issue with the product, stop using it and contact the customers center.
- Children under the age of 7 should be used under the guidance and supervision of a guardian.

FAQ

Questions	Answers
The power does not turn on.	If the power does not turn on even with fully charged, please contact the customer service center.
Device doesn't operate for operating hours for each mode as stated even with fully charged.	If the battery life (based on about 300 times of charging) is exceeded, the operating hour may be shortened. Please contact the customer service center for the battery replacement.
No PEMF stimulation is felt.	PEMF used in Ikeun™ is a low-frequency electromagnetic field of approximately 9-28 Hz, so you don't feel pain.
I want to adjust the vibration intensity.	Vibration intensity cannot be adjusted. If vibration stimulation is uncomfortable, please choose the sleep mode.
Can I use it while taking a bath?	This product is not waterproof. Do not immerse or touch water.
Can I use it on other body areas rather than knee?	This product is designed to stimulate growth plate of the knee area. Please use properly as stated in user manual.
How often can I use it in a day?	It is recommended that you can use either sleep or intense mode once a day.

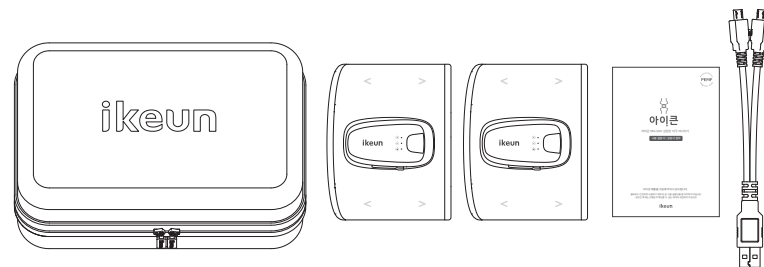Product Specification

MIK-500-005EN

Product name	Growth Plate Stimulator ikeun™
Model name	MIK-500
Body size (1EA)	W480 x H110 x D30 mm
Body weight (1EA)	190g
Configuration	Main body 2EA, Y-type cable 1EA, Pouch 1EA, User manual
Certification	KC (R-R-mk1-HNB1-MIK500) FCC CE
Battery safety testing report number	XU100782-17002F
Product rating	3.7V, 500mAh
Power Consumption	0.8W
Charging time / usage time	A full recharge requires up to 3 hours
Use environment	10°C~35°C
Country of origin	Republic of Korea
Manufacturer	MN Co. Ltd.

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